

COXO[®]

Professional Dental Manufacturer

SINCE 2003

C-Root I(V)

Root Apex Locators

Simple interface and Accurate positioning



COXO[®]

Professional Dental Manufacturer

SINCE 2003

Long Battery Life

1200mAh

Battery capacity

≥5.5h

Continuous operation



C-Root I(V)
Root Apex Locators

Display the file position in the root canal





A Insert the file into the root canal.


The apical distance between the root tips shows a changing trajectory as the file advances, and the value of the root canal length indicator bar also changes accordingly.

B When the file tip reaches the apical foramen, the root canal length indicator bar begins to display and emits a low-frequency "beep", "beep" prompt sound.

C When the file tip reaches the apical foramen, the alarm sound changes to a tight and high-frequency "beep ~" prompt sound.

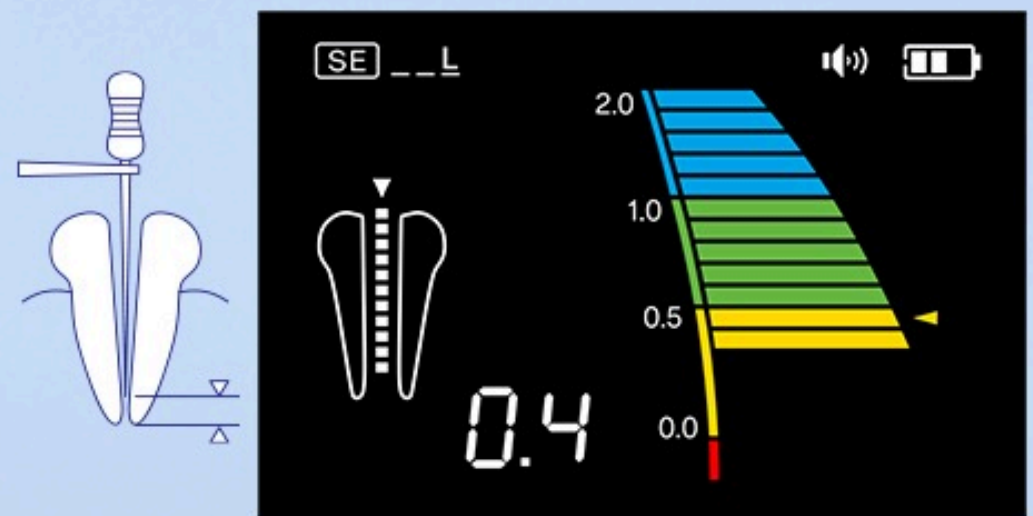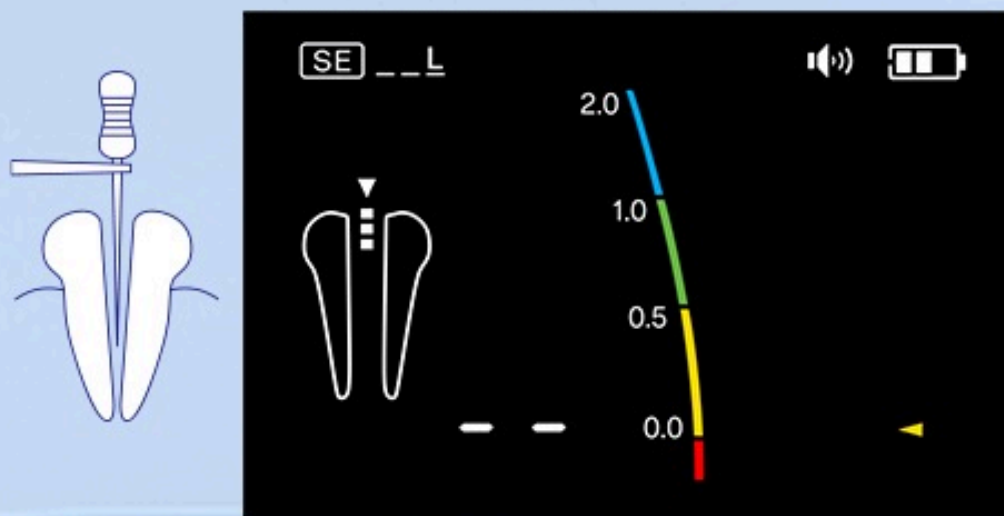

D The value "00" indicates that the apical hole has been reached, and when the file advances into the red area of the Apex, the tip of the file is beyond the apical hole.

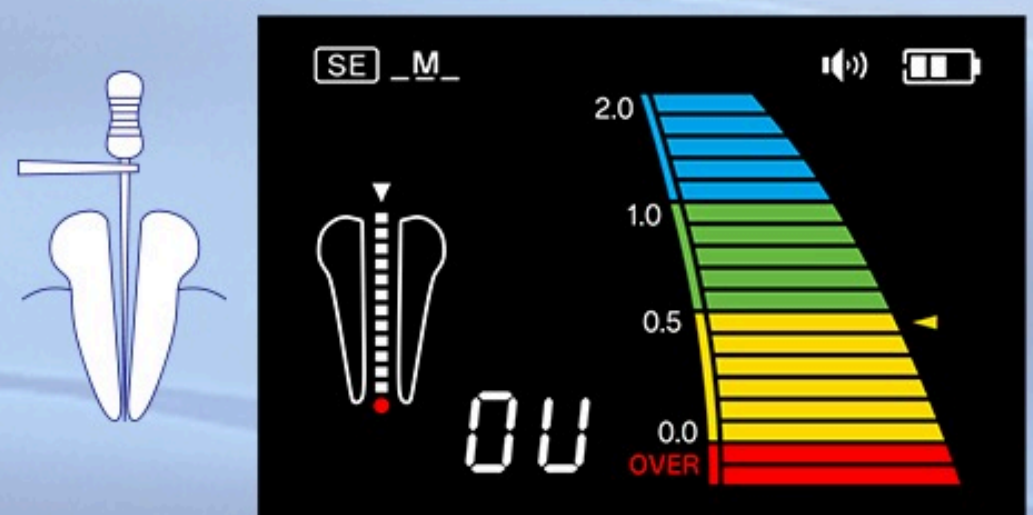

COXO[®]

Professional Dental Manufacturer

SINCE 2003

A Root test sensitivity setting:
High, Medium, Low

B Short circuit test prompt icon:
File clip and lip hook short circuit
display normal

C Complete root canal image

D Root canal length indicator strip :
Apex positioning position display

E 0.0 APEX apical foramen:
The "Over" area indicates that the
file exceeds the apical foramen


F Power switch



G Alarm volume setting:
OFF, Low, Medium, High

H LCD display:
Visible under strong light

I Type-C measuring cable/
power interface

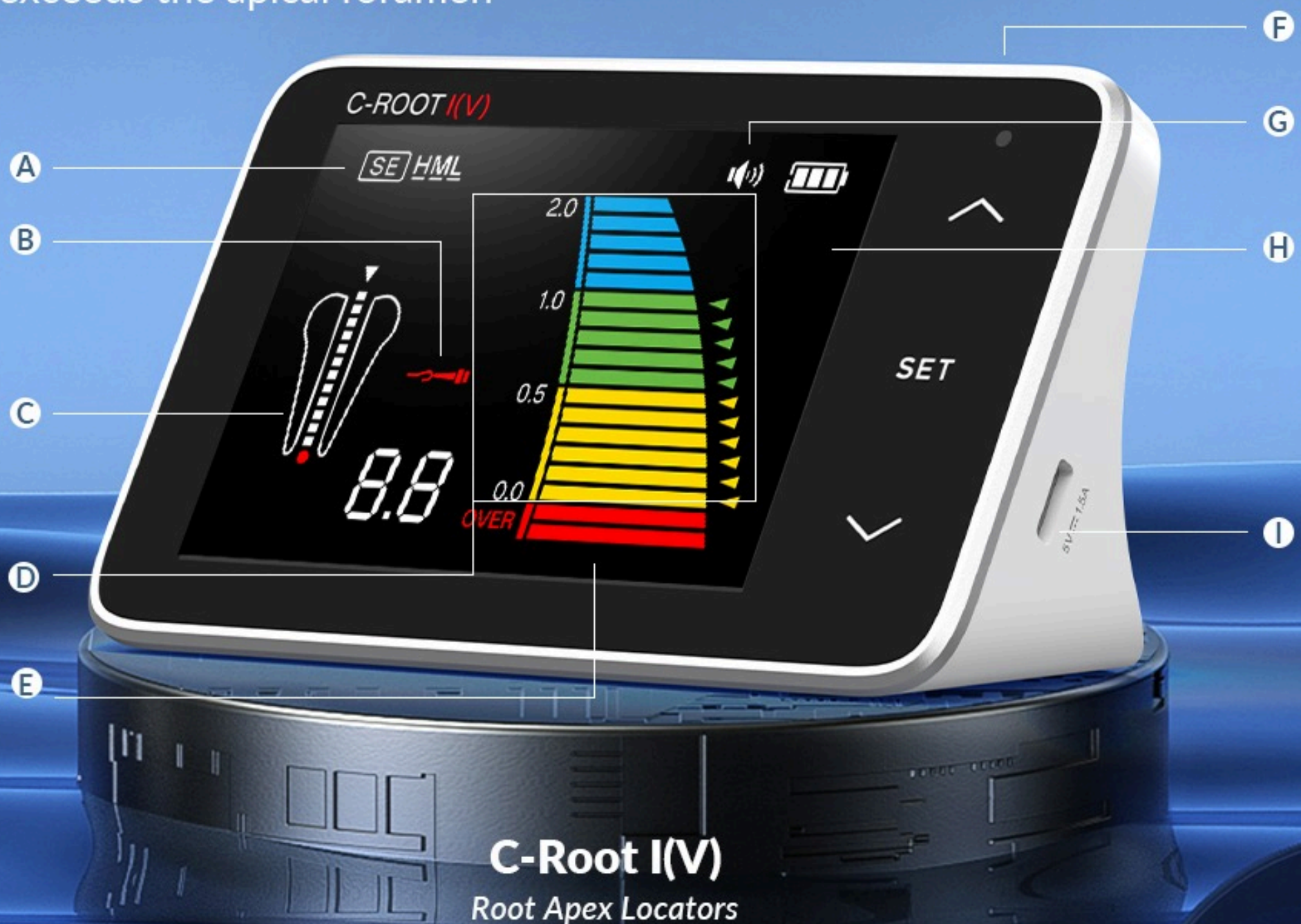

C-Root I(V)
Root Apex Locators